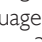
EN Insert four AA size alkaline batteries in the battery compartment in the back of the AED Trainer 3.

  Press and hold both buttons for 6 seconds. Release both buttons, then press the  button to scroll through list of languages until the desired language is announced. To save the language selection, turn off the AED Trainer 3 by pressing the  button. The language will be active the next time you turn on the AED Trainer 3.

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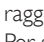
Apply "TRAINING ONLY" sticker for the selected language.

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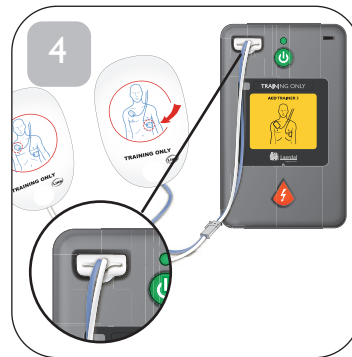
Plug the Training Pads connector into the connector port on the front of the AED Trainer 3. Prepare the manikin to accept pad placement as appropriate. You are now ready to start training.

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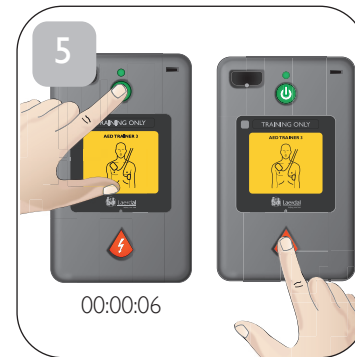
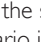
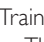
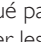
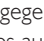
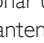
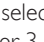
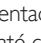
To select a different scenario, press and hold the  button for 6 seconds. Press the  as directed to scroll through the scenarios until the desired scenario is announced. To save the selected scenario, turn off the AED Trainer 3 by pressing the  button. The scenario will be active the next time you turn on the AED Trainer 3.

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- A  C 
B  D 

EN Scenario codes

- A = Shockable Rhythm
B = Non-shockable Rhythm
C = Pads problem detected
D = CPR First protocol initiated

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AED Trainer 3

Quick Setup Guide

© 2022 Laerdal Medical AS. All rights reserved.

 Manufactured in China for Laerdal Medical AS,
P.O. Box 377,
Tanke Svilandsgate 30,
4002 Stavanger, Norway

20-03295 Rev E

www.laerdal.com