


SimJunior

Quick Setup Guide

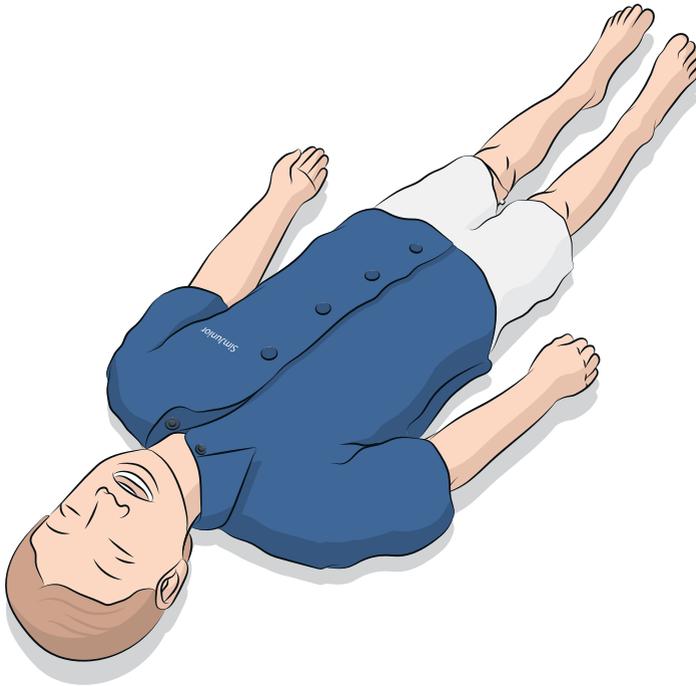


This Quick Setup Guide describes how to assemble and setup your SimJunior patient simulator. For information about how to use SimJunior, please refer to LLEAP help files and SimJunior Directions for Use.

Visit www.laerdal.com/downloads to download the latest Directions for Use and software.



INCLUDED COMPONENTS

Simulator

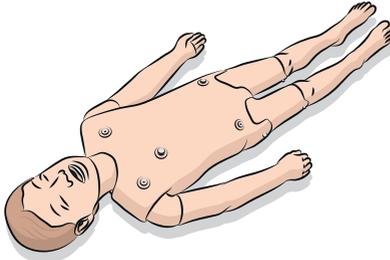

SimJunior Clothing



Manuals & Warranty



Hardware Kit



BP cuff



Eye Set



IVR IV bag



SNB IV bag



9V Power Supply



Lubricant



Shoulder Adaptor



Blood Concentrate



Baby Powder



Modelling Wax



REQUIRED

Instructor PC



or
SimPad



Link Box



Power adaptor



Ethernet cable for PC



RECOMMENDED

Patient Monitor



Headset



Ethernet switch



SpO2 probe




Ethernet cable for
Patient Monitor




1

CHARGE BATTERIES


Instructor PC

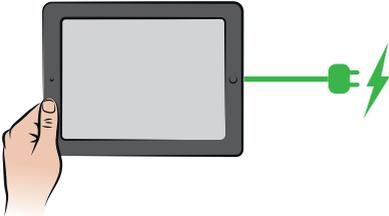


SimPad

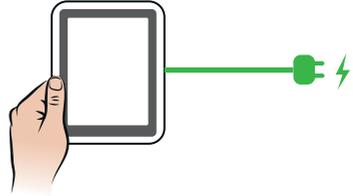

Patient Monitor

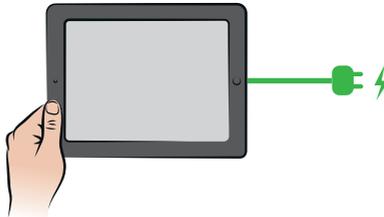


Link Box

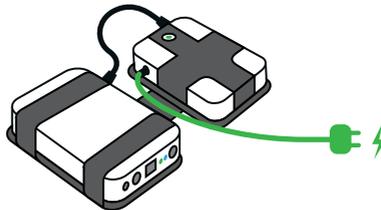


2A


WIRED SETUP

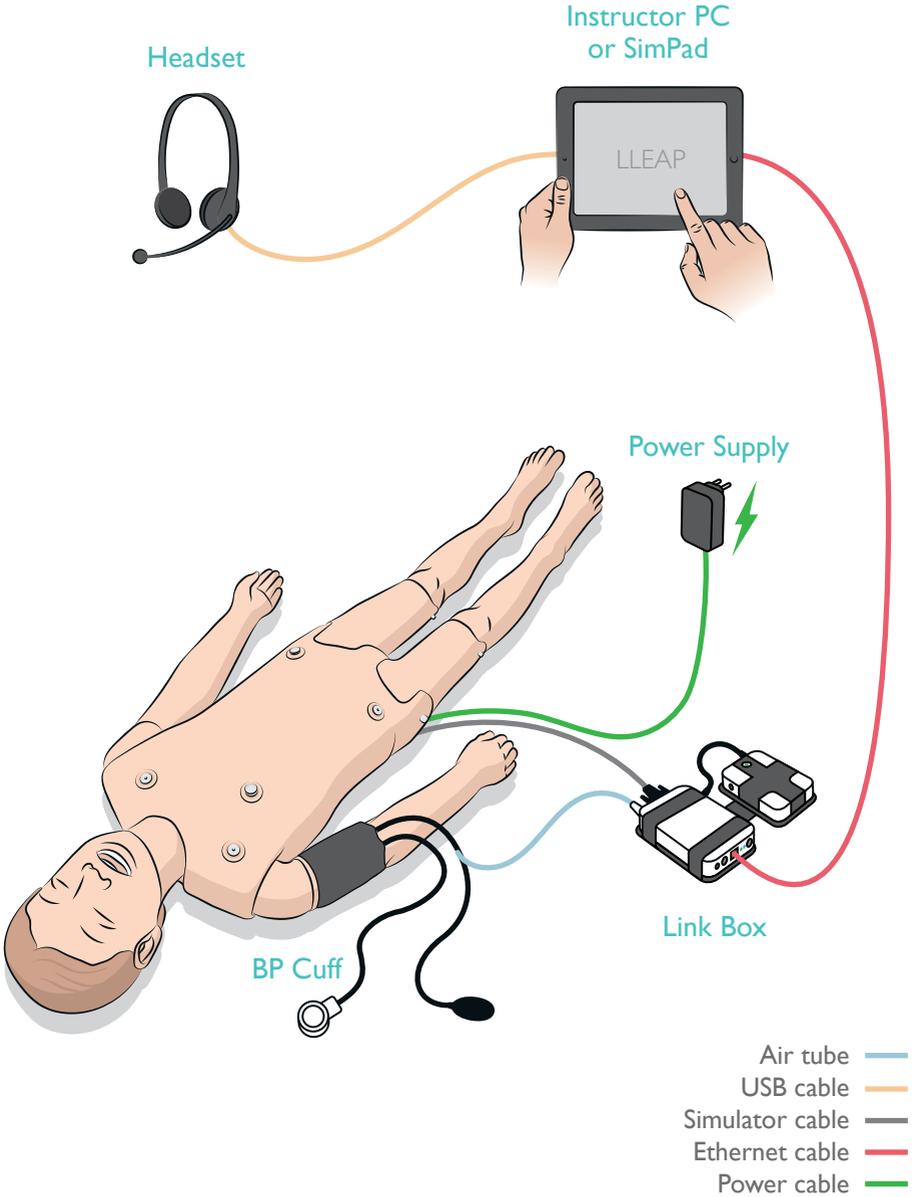

2B


SIMPAD WIRELESS SETUP

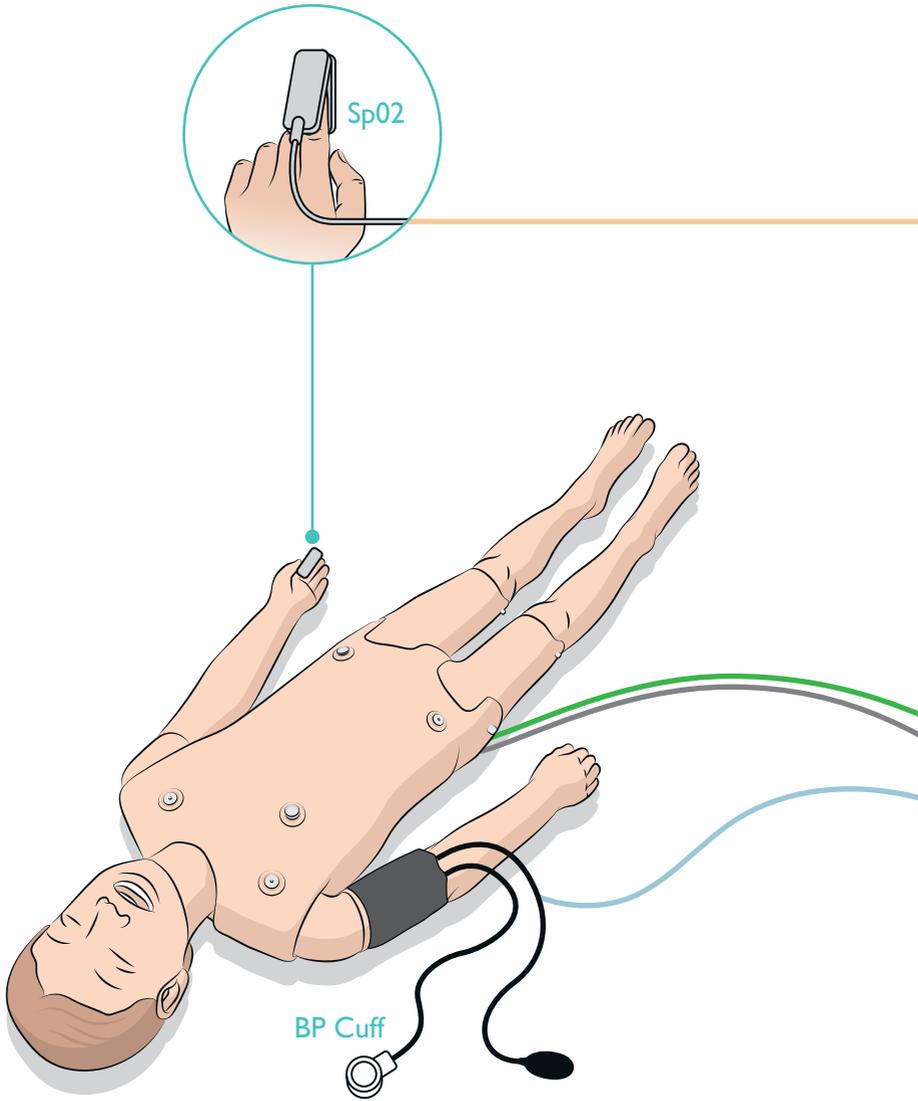


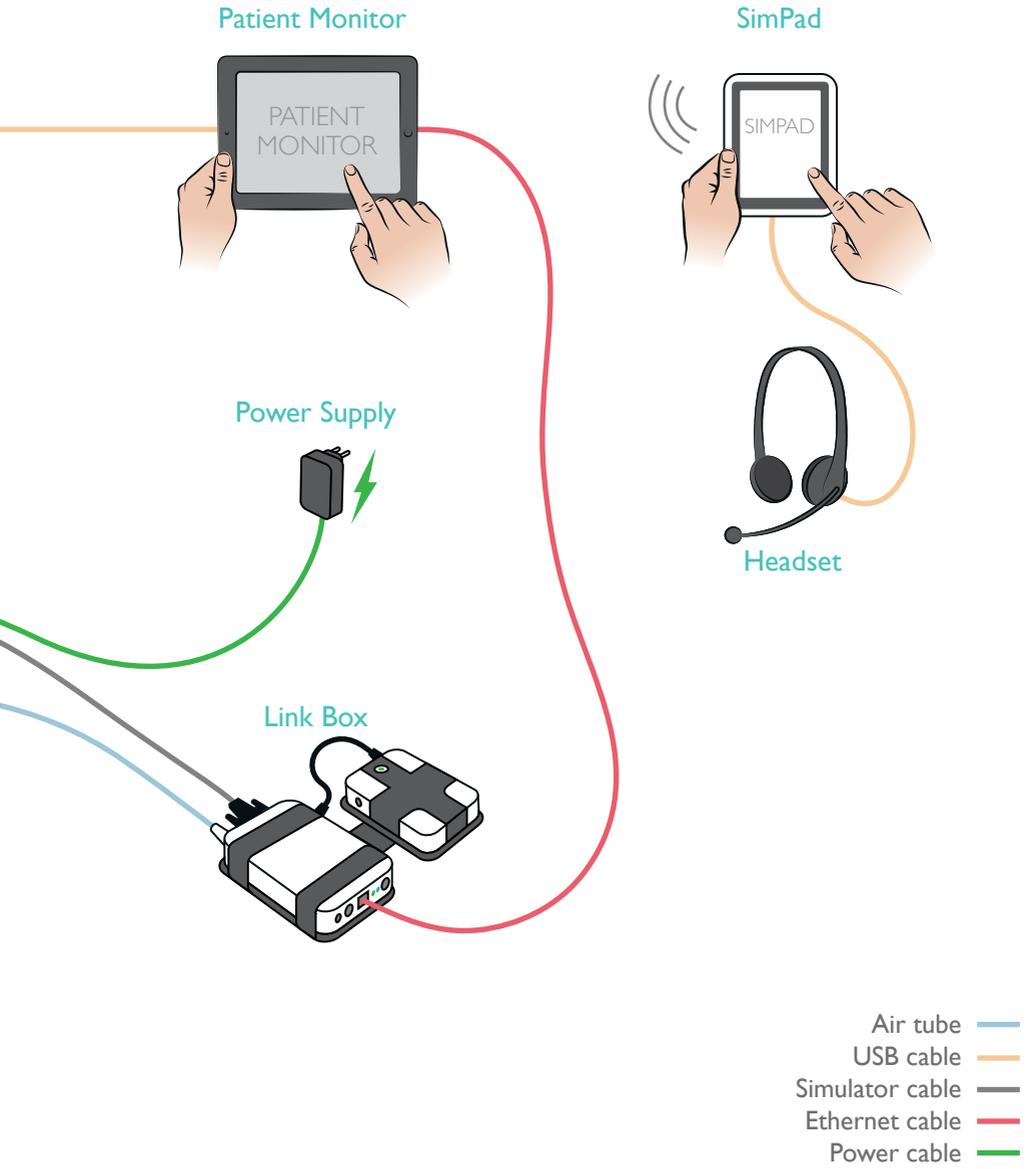

2c

WIRED SETUP WITH PATIENT MONITOR

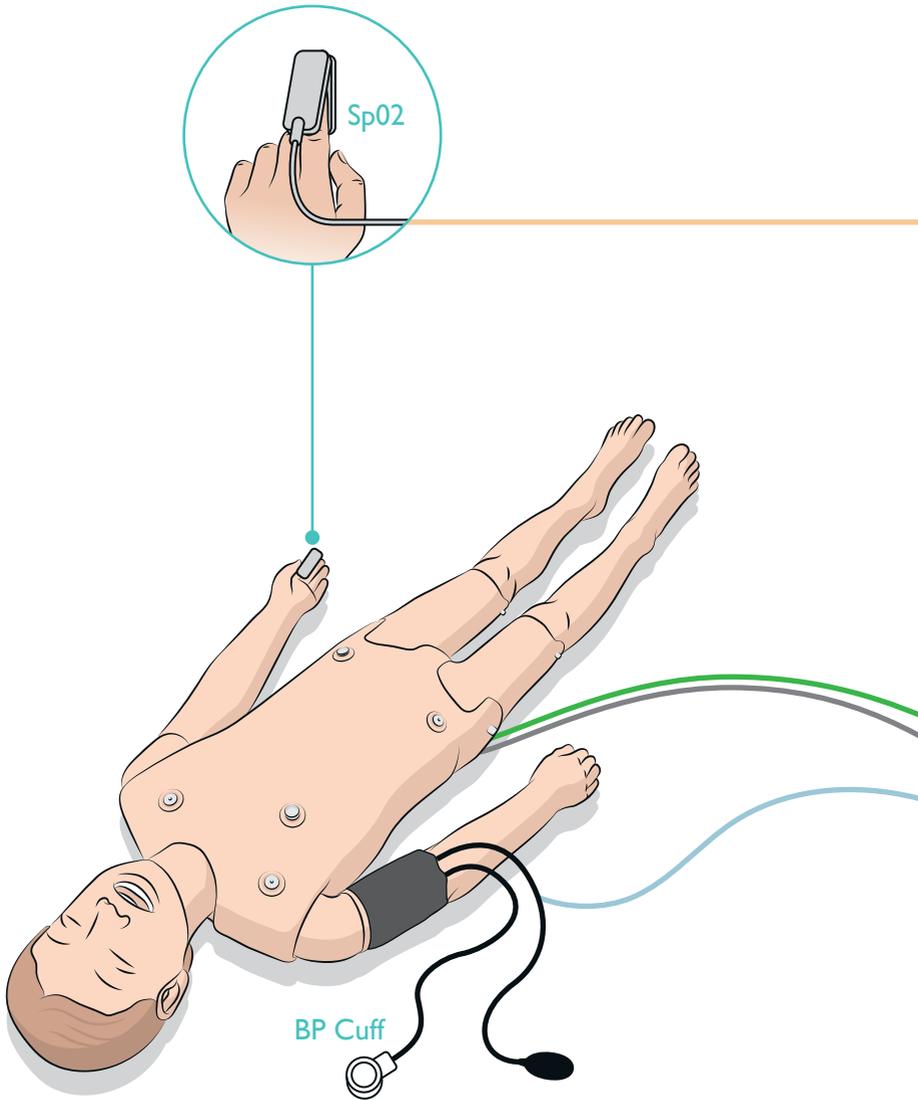

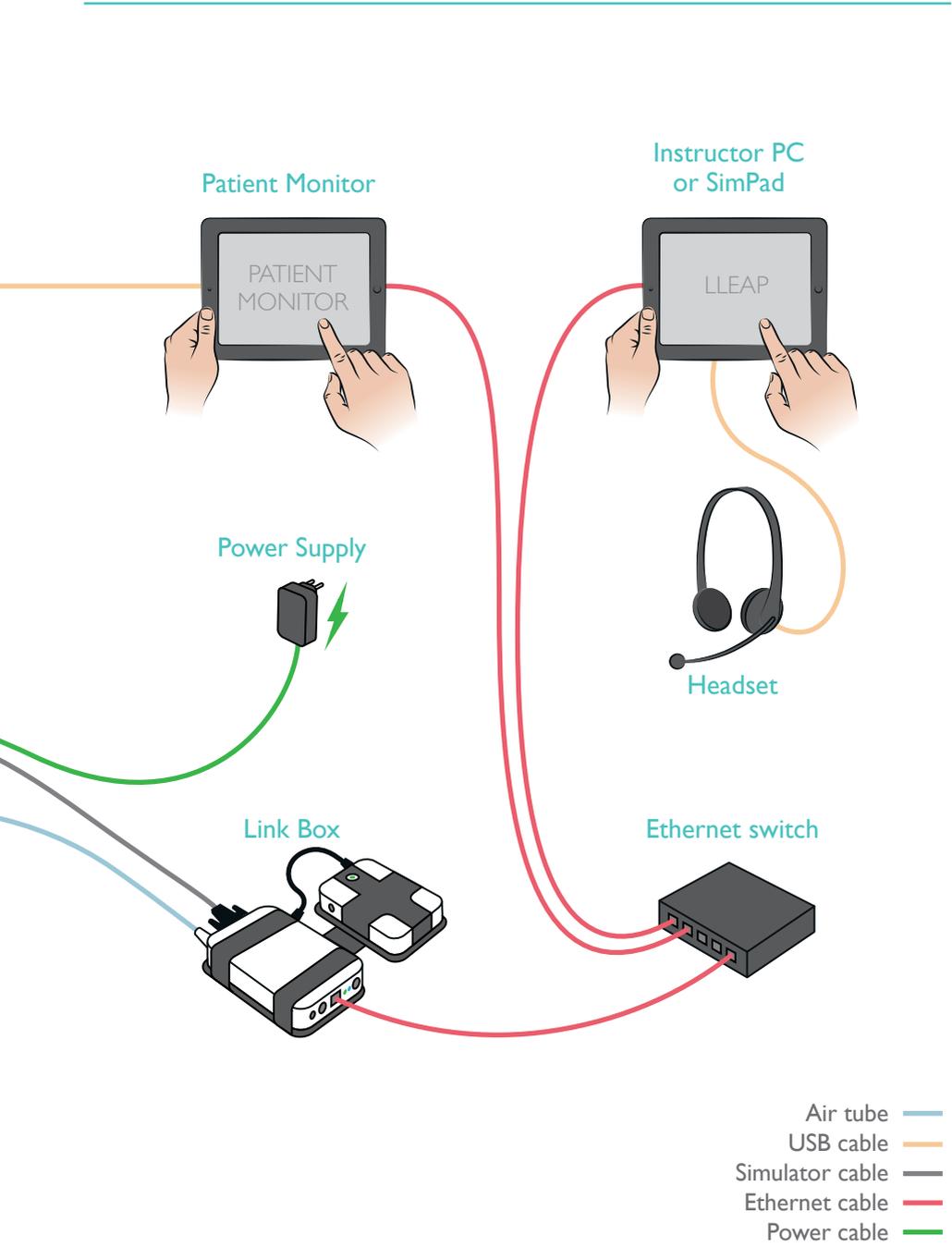

© 2014 Laerdal Medical AS. All rights reserved.
Device Manufacturer: Laerdal Medical , P.O. Box 38,
226 FM 116, Gatesville, Texas 76528 USA

Rev A



www.laerdal.com



Laerdal
helping save lives