


SimBaby - Preventative Maintenance Lite Kit

Before you install this Preventative Maintenance Lite Kit, please go to [SimBabyVideo](#) and watch the video instructions.

Items Included

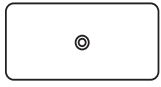

Stomach Bladder




Lung Bladder



Stomach Foam




Liver Bladder

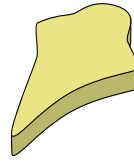

Fontanelle Foam



Tape

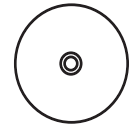

Fontanelle Bladder

Remove

1. Remove the arms, legs and skin, and set aside.
2. Disconnect the Liver Bladder and discard.
3. Disconnect the Stomach Foam and Bladder and discard.
4. Remove the Fontanelle Foam and Bladder and discard.

Replace

1. Replace Stomach Foam.
2. Replace Stomach Bladder.
3. Replace Lung Bladders.
4. Replace Liver Bladder.
5. Replace Fontanelle Bladder.
6. Secure Fontanelle Foam to Bladder with double-sided tape.
7. Replace the skin, arms and legs.

Test

Connect to the SimBaby with a SimPad and check the functionality of:

- Fontanelle bladder
- Liver bladder

Using a Bag-Valve-Mask (BVM) check that the lungs inflate as expected and that the Stomach Distention bladder inflates when too much air is forced into the lungs.

© 2021 Laerdal Medical AS. All rights reserved.

Manufacturer: Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway
T: (+47) 51 51 17 00, Printed in Norway

www.laerdal.com