


Nursing Anne Simulator - Preventative Maintenance Kit

Before you install this Preventative Maintenance Lite Kit, please go to [Nursing Anne Simulator](#) and watch the video instructions.

Items Included

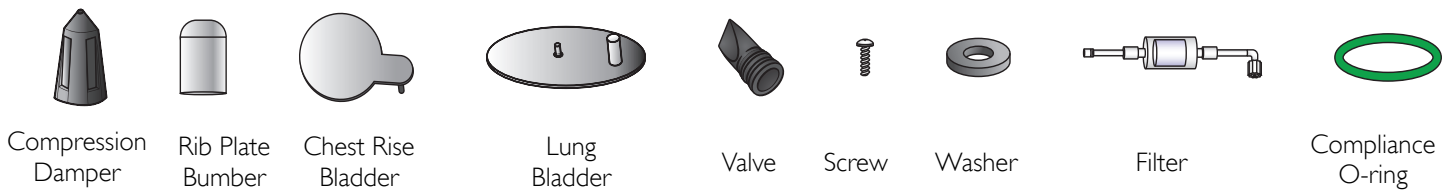

Remove

1. Remove the plug that attaches the Torso skin to the stomach pad.
2. Peel off the Torso skin.

Replace

Replace the valve assembly in the genitalia

1. Remove the genitalia assembly.
2. Disconnect the urine bag (from where).
3. Remove the two black screws (where are they? What are they holding?)
4. Remove the valves located inside the white screw assembly.
5. Place the new valves inside the white screw assembly.
6. Place and tighten the two black screws (on what?)
7. Connect the urine bag (to what).
8. Place the genitalia assembly back and make sure it is placed on its securing magnet.

Replace the Rib Plate Bumper

1. Remove the stomach without disconnecting any tubes.
2. Lift the stomach off the chest.
3. The rib plate bumper is located under the top rib plate.

Replace the green silicone O-ring

1. Remove the o-rings attached on each side of the chest plate.
2. Place the new o-rings.

Replace the Chest Rise Bladder

1. Remove the chest rise bladder located under the white chest rise levers.
2. Place the new chest rise bladder.

Replace the Lung Bladders

1. Remove the lung bladders located between the black and white chest plates.
2. Place the new lung bladders.

Replace the Compression Damper

1. Remove the compression damper located under the compression spring.
2. Place the new compression damper.

Replace the Dryer Filter Assembly

1. Remove the dryer filter assembly located under the white fluid protector sheet.
2. Place the new dryer filter assembly.

Reassemble

1. Put the Torso skin back on.
2. Connect the plug to attach the torso skin to the stomach pad.

© 2022 Laerdal Medical AS. All rights reserved.

Manufacturer: Laerdal Medical AS

P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway

T: (+47) 51 51 17 00, Printed in Norway

www.laerdal.com