

https://scenariocloud.laerdal.com



Scenario Library

My Scenarios

Karen Howard ▾

Scenario Library

Discover content that has been chosen to support your simulation journey.

Search

🔍 covid



Filters

Medical Specialities

All Medical Specialities ▾

Nursing Specialities

All Nursing Specialities ▾

Nursing Courses

All Nursing Courses ▾

Body Systems

All Body Systems ▾

Showing 1-6 of 6 results

Sort by: Newest First ▾ Title A-Z ▾

Starter

Normal Childbirth with Symptomatic Covid-19

LAERDAL MEDICAL

Starter

Covid-19 Part 3: Proning Procedure of Patient on a Ventilator

LAERDAL MEDICAL

Starter

Covid-19 Part 2: Inspiratory Dyssynchrony on Ventilator

LAERDAL MEDICAL

Starter

Covid-19 Part 1: Remission

Starter

Infection Prevention and Control Scenario 1

Starter

Infection Prevention and Control Scenario 2






Laerdal

helping save lives



Transferring Themes to SimPad

1. Download Theme to desktop
2. Connect SimPad to computer with supplied USB cable
3. SimPad 'opens' on computer
4. Navigate to Themes folder
5. Copy/paste or drag Theme to Themes folder

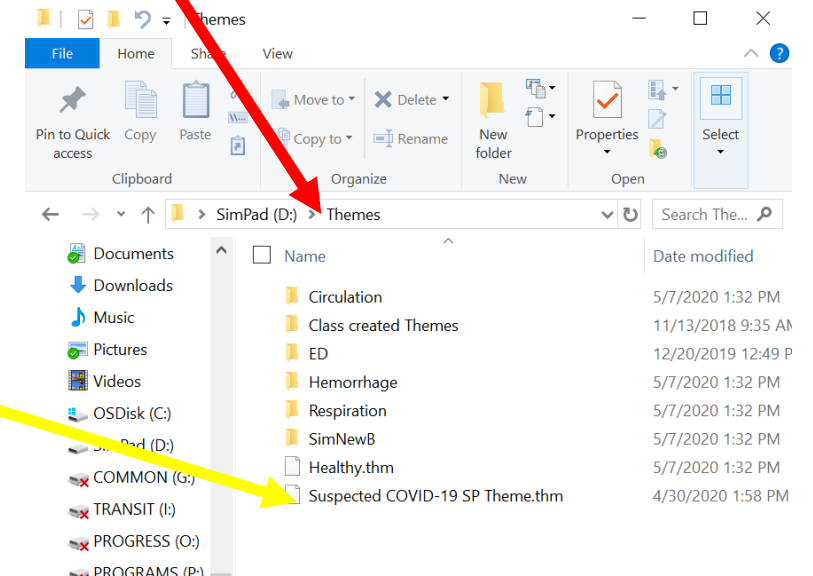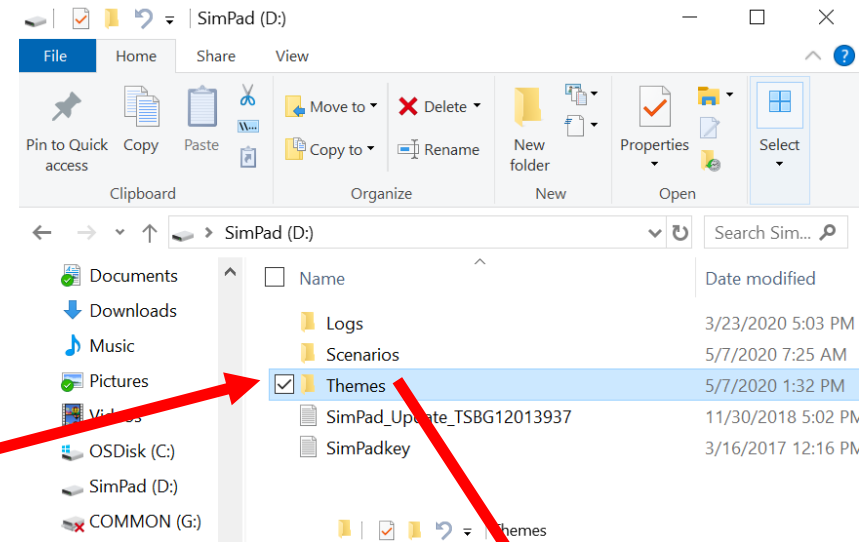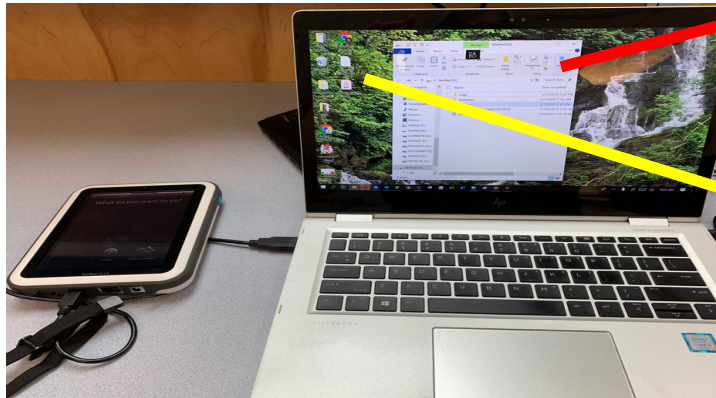

Transferring to LLEAP

1. Download Theme to desktop
2. Open LLEAP and navigate to Folder Path Editor Tile
3. Open Folder Path Editor Tile and locate Theme files
4. Move to the right and Browse to open Theme folder
5. Copy/paste Theme from desktop to Theme folder

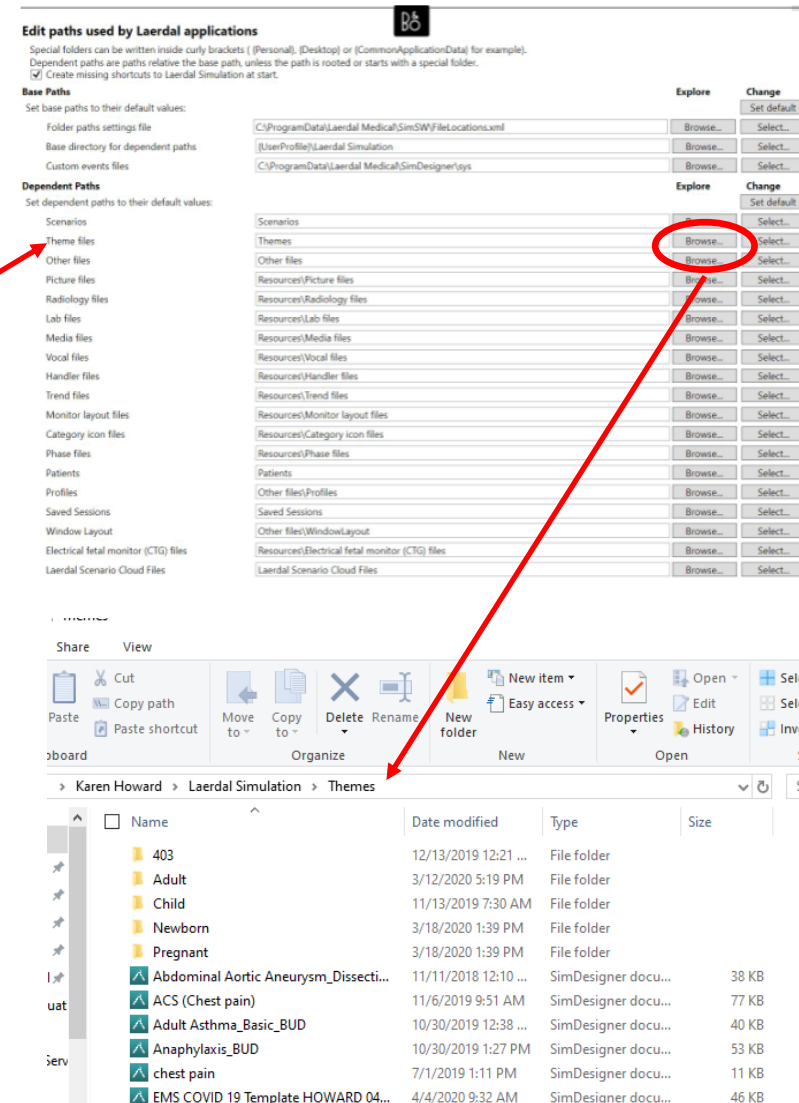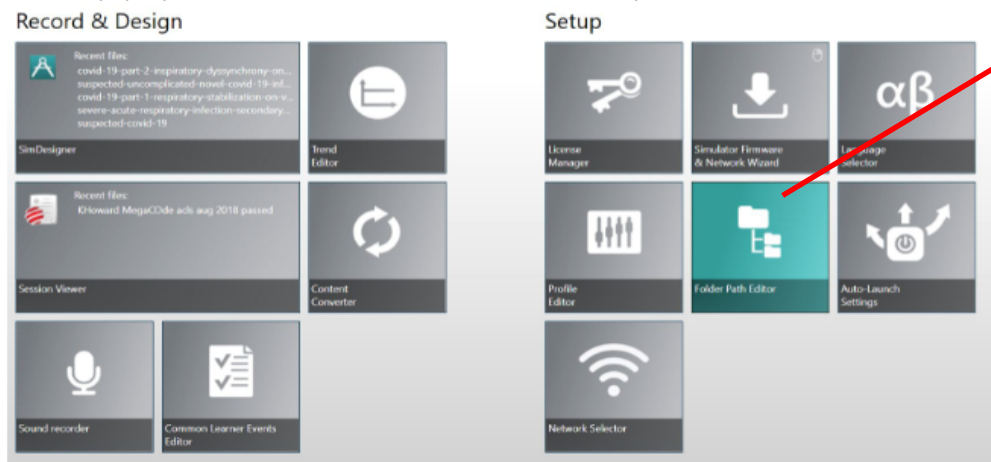

The process is similar for adding scenarios, vocals, radiology Files, pictures, etc,

Karen Howard, BSN, RN, CHSE
Karen,howard@Laerdal.com



Laerdal

helping save lives