



Nursing Kid - Preventative Maintenance Lite Kit

Before you install this Preventative Maintenance Lite Kit, please visit www.laerdal.com/laerdalpmkit and watch the instructions.

Items Included

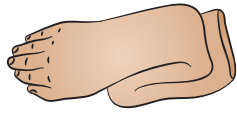

Arm skin

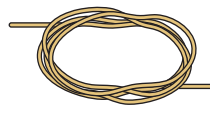


Veins



Lungs



Lung securing
bands

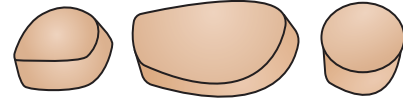

Pads

Replace the Intra Muscular (IM) pads

Replace the IM pads inside the shoulder, lower back and thighs.

Replace the lungs

1. Remove the chest plate and set it aside.
2. Loosen and remove the screw holding the Head and set it aside.
3. Pull on the head to access the airway tubes.
4. Place the elastic bands on the airway tubes.
5. Take the new lungs.
6. Connect the lungs to the airways and secure them with the elastic bands.

Mount the head

1. Push the head back in its place.
2. Flatten the lungs on the torso.
3. Screw the head back on the torso.
4. Close the chest plate.

Replace the right arm skin and veins

1. Carefully cut the right arm skin and peel it off.
2. Remove the veins.
3. Clean the arm to remove any dust or left over material.
4. Place the new veins in the hand and between the fingers.
5. Apply small amounts of super glue where the veins touch the hand.
6. Press and hold the veins in place until the glue sets.
7. Repeat the action on the other side of the arm.
8. Lubricate the inside of the new arm skin with liquid soap.
9. Put the arm skin on the manikin's right arm.

 Note

If needed, apply more liquid soap to make the arm skin slide on more easily.