



SimNewB - Preventative Maintenance Lite Kit

Before you install this Preventative Maintenance Lite Kit, please go to [SimNewB Video](#) and watch the video instructions.

Items Included

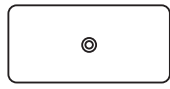


Bladder 40x90mm
Stomach Distention

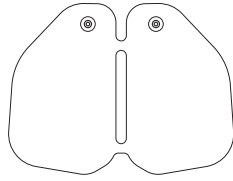


Lung Bladder



Tape

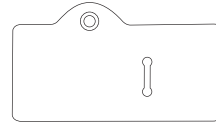

Chest Rise Bladder

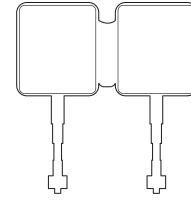

Lung Sound Bladders
Assy

Dissassembly

1. Remove the arms, legs and skin, and set aside.
2. Remove the skin and set aside.
3. Disconnect the Stomach Distention Bladder at the tube connection where it enters to Stomach Foam. This bladder will be replaced during this PM.
4. Disconnect the Lung bladders, left and right.
5. Disconnect the Chest Rise bladder on the left side.
6. Lift the Chest Plate away, but do not disconnect the ribbon cable or the strings holding the chest plate in place.

Chest Bladders

1. Remove the chest rise bladder (closest to the stomach foam). Note that the bladder is folded in half, with the fold closest to the head. Replace with new bladder. There are 3 pieces of double sided tape used here; on each plate and in-between the bladder (after it is folded).
2. Replace double sided tape on the bladders and chest plate and press together.
3. Connect Speaker bladders (now connected to rest of chest assembly) to connectors at the shoulders.
4. Place foam over umbilical chord.
5. Replace chest assembly over foam.
6. Reconnect Lung, Chest-rise and Stomach bladder tubes.
7. Replace Torso skin and reattach arms and legs.
8. Follow and fill out the SimNewB Final Service Checklist.

© 2021 Laerdal Medical AS. All rights reserved.

Manufacturer: Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway
T: (+47) 51 51 17 00, Printed in Norway